

# IVAN | MED<sup>M</sup>

AESTHETIC EQUIPMENT  
SKINCARE | TRAINING



CryoPen<sup>®</sup>

**CryoPen<sup>®</sup>**

More Precision, More Performance



## Easy to use

Even if you have had no previous experience performing cryosurgery, all you need are your diagnostic skills and a steady hand.

CryoPens® pen like shape allows for easy handling and for treatments to be performed in hard to reach, smaller places like around the eyes.

## Perfect removal of skin lesion of any shape or size

CryoPen® is supplied with a range of applicators allowing for change of tip sizes dependant on the shape and size of the lesion being treated.



**Blue/Red dot applicator**

Ø 1-3mm flow time up to 250sec



**White/Red dot applicator**

Ø 2-6mm flow time up to 180sec



**Green/Red dot applicator**

Ø 4-8mm flow time up to 145sec

CryoPen® can be used with 2 sizes of cartridges (8g or 16g) with up to 15 treatments possible from one cartridge. Once punctured, cartridges will remain without loss of pressure for many weeks.



## What is Cryosurgery?

Cryosurgery is an advanced freezing innovation that is fast, effective and safe for the removal of benign skin lesions on the face and body.

CryoPen® Cryosurgery device delivers an ultra-cold jet of high-pressure nitrous oxide directly on to skin lesions. The nitrous oxide destroys the skin tissue by freezing the intercellular fluid and forming ice shards what rupture the cells membrane, destroying the cells. The ultra-cold nitrous oxide cuts off the blood supply during the freezing process, ensuring eh client's safety during the process.

## Perfect removal of skin lesions

CryoPen® delivers exceptional results for the complete removal of all benign skin lesions in just one treatment session.



### Acrochordon

Treatment:  
10-15 seconds



### Seborrheic Keratoses

Treatment:  
5-10 seconds



### Pigmented Spot

Treatment:  
3-5 seconds



### Solar Lentigo

Treatment:  
3-5 seconds



### Molluscum Contagiosum

Treatment:  
10-15 seconds



### Haemangioma

Treatment:  
5-10 seconds



### Skin Tag

Treatment:  
5-10 seconds



### Wart

Treatment:  
10-15 seconds

## Treats all benign skin lesions

CryoPen® treats all common skin lesions in seconds including verrucae, skin tags, solar lentigo, pigmented spots, actinic keratosis, seborrheic keratosis, haemangioma, molluscum contagiosum, condyloma, plantar warts and more.

## Comfortable, fast, no downtime



CryoPen® requires minimal aftercare post treatment.

## Results in as little as one treatment

**Before treatment**

**During treatment**

**After treatment**

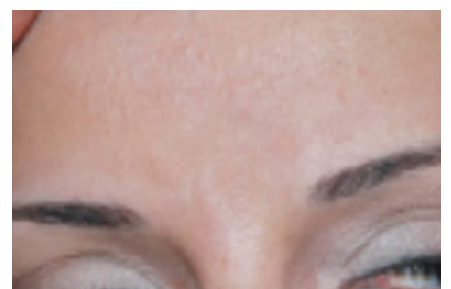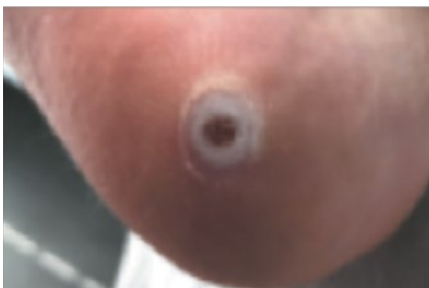

## Leading cryosurgery

CryoPen® is a range of innovative, powerful, handheld cryosurgery tools, designed to completely remove all benign skin lesions.

CryoPen® is available in two models – CryoPen® M & CryoPen® O both suitable to remove benign skin lesions. The CryoPen® M is designed to allow aesthetic practitioners to treat superficial skin lesions and comes with 2 applicators. CryoPen® O allows the treatment of larger lesions such as warts and verrucae.

The CryoPen® M & CryoPen® O can be used by dermatologists, aesthetics practitioners, podiatrists and surgeons.

**In November 2018 the NHS released a statement removing the treatment of benign skin lesions from GP surgeries within the UK making the CryoPen® a must have treatment for any clinic.**

## Benefits of CryoPen®

### **Extreme precision**

Microfine just for precise control making it safe to use on delicate skin or around the eye area

### **Extreme power**

N<sub>2</sub>O at high pressure (55 bars) and low temperatures (-89 degrees) for effective cell destruction

### **Improved treatments**

Comfortable treatment, no stress and minimal damage if any, to existing tissue

### **Improved results**

Impressive results for complete removal of benign skin lesions

### **Cost effective**

Efficient, practical and extremely cost effective for a quick return on investment

## In the package

### CryoPen® M

- CryoPen® M device
- 2 applicators
- 24 x 8g nitrous oxide cartridges
- Tweezers for inserting cartridges
- User manual
- Print marketing – information flyers, FAQ cards, posters and consultation forms
- Digital marketing – before and after images, videos, product shots and social media images

### CryoPen® O

- CryoPen® O Device
- 3 applicators
- 12 x 16g cartridges
- Tweezers for inserting cartridges
- User manual
- Print marketing – information flyers, FAQ cards, posters and consultation forms
- Digital marketing – before and after images, videos, product shots and social media images



**0333 358 3904**

info@ivanmed.com

ivanmed.com

### Training & Insurance

All our CryoPen® packages are offered along with fully accredited OCN CryoPen® training at one of our dedicated training centres across the UK.

CryoPen® theory training is carried out online with practical training offered at one of our training venues. All our CryoPen® training sessions can also gain CPD points as part of the training.